

# MTA



MG InHex® / MG Osseous®  
Implant

MTA® Final  
Abutment

+

Cover Screw &  
Prosthetic Screw

+

MTA® Impression  
Coping

## New MTA®

### This new MTA line offers the easiest and most effective solution

The implant (MG InHex® o MG Osseous®) together with the MTA abutment with implant mount function allows the maximum use of all its prosthetic components with the high quality has always characterized Mozo-Grau

This new Line presents, together with a Mozo-Grau implant, a final MTA® abutment, an impression coping, a cover screw and a prosthetic screw, which makes it into the optimal solution to achieve the maximum efficiency of all its components.

**MOZOGRAU®**  
D E N T A L I M P L A N T S

## Why is **MTA**® The Optimal Solution?

- Because it reduces the number of components needed to perform a successful restoration ensuring all its functionalities.
- Because it is a unique solution. All **MTA**® components are fully maximized to cater with the professionals' requirements.
- Because it is a cost-effective solution that copes with now-a-days clinicians' and their patients' needs.

## Instructions for use



Pick up the implant from the vial using the MTA driver



Carry safely the implant from the vial to the patient's mouth



Insert the implant in the alveolus



Unscrew the MTA



Remove the MTA and place the cover screw/healing abutment



Remove the security ring  
STEP 1



Remove the security ring  
STEP 2



Remove the security ring  
STEP 3

## Impression Taking



Screw the MTA



Place the MTA impression coping



Pour silicone into the impression tray



Seat the impression tray in the mouth



Remove the impression tray with the MTA copings inserted in it. Unscrew the MTA



Screw the analogue to the MTA



Place the MTA and the analogue into the impression coping



Pour plaster in the impression tray



Cast the model



View of the MTA with the analogue in the lab model

## **MTA**® advantages

- It offers both the benefits of the implant mount and the prosthetic abutment in one single product.
- It offers all the components needed to make a restoration. It is no longer necessary to re-use any element.
- There is no need to modify any of its components since they fit perfectly. They have been designed to perform the tasks they have been developed for.
- Thanks to the **MTA**® abutment and impression coping flat side, and its click system it is easy to verify the accurate fit between them to achieve a successful restoration.

### Headquarters

C/ Santiago López González, 7· 47197 Valladolid · Spain  
Tel. +34 983 211 312 · Fax +34 983 304 021· info@mozo-grau.com

[www.mozo-grau.com](http://www.mozo-grau.com)

Follow us:



**MOZOGRAU**®  
D E N T A L I M P L A N T S